First EC marking date applied : 2014

Manufactured in France by  
TEKNIMED S.A.S  
8, rue du Corps Franc Pommiers  
65500 Vic en Bigorre - FRANCE  
Tél : (33) 5.62.96.88.38  
Fax : (33) 5.62.96.28.72  
www.teknimed.com

TF410609/3 \_ 09/2019

**This equipment must only be used for injectable PMMA bone cements during arthroplasty procedures.**

This device is packaged and sterilized for single use only. Do not reuse, reprocess or resterilize. Reuse, reprocessing, or resterilization may compromise the structural integrity of the device and/or lead to device failure that in turn may result in patient injury, illness or death. Also, reprocessing or resterilization of single use devices may create a risk of contamination and/or cause patient infection or cross-infection, including, but not limited to, the transmission of infectious disease(s) from one patient to another. Contamination of the device may lead to injury, illness, or death of the patient.

Comply with local regulations in force relating to medical waste for safe handling and disposal of the cement.

**(A)** Injection handset  
Pistolet d'injection  
Injektionspistole  
Pistola de inyección  
Pistola da iniezione  
Pistola de injeção  
Enjeksiyon tabancası

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
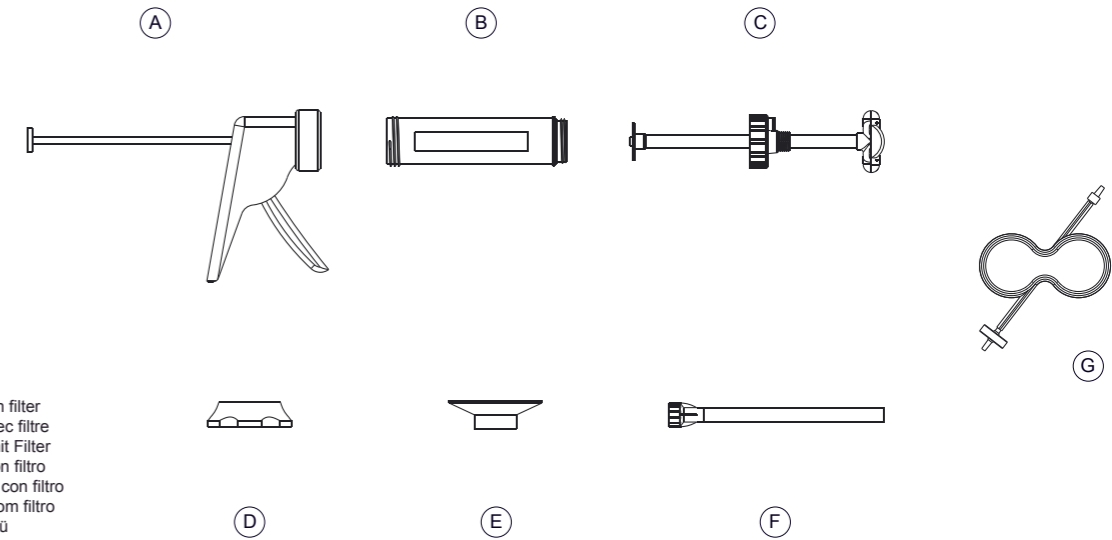
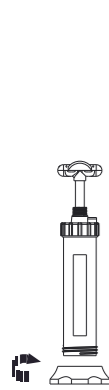
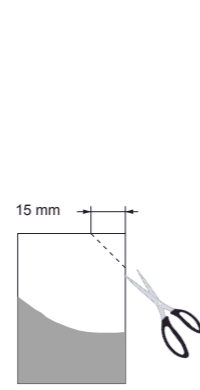
Remove the mixer plunger.

④

Cut one corner from the pouch of bone cement powder.

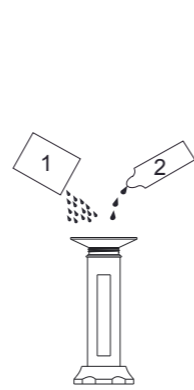⑤


Place the funnel in the reservoir and pour in the bone cement powder first, followed by the liquid.

⑥


Screw the mixer plunger onto the reservoir.  
Caution: check that the mixer plunger's impeller is fully home at the base of the reservoir.

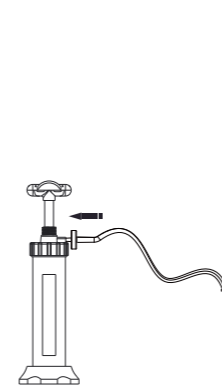⑦


Firmly connect the vacuum tube to the mixer and the adaptor to a vacuum pump pedal.

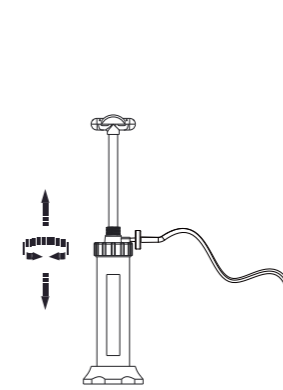⑧

Mix by back-and-forth motions whilst simultaneously rotating the mixer plunger for the time required (refer to the bone cement's user instructions).  
Caution: keep the mixer vertical and do not remove the base as long as the mixing phase is not completed.

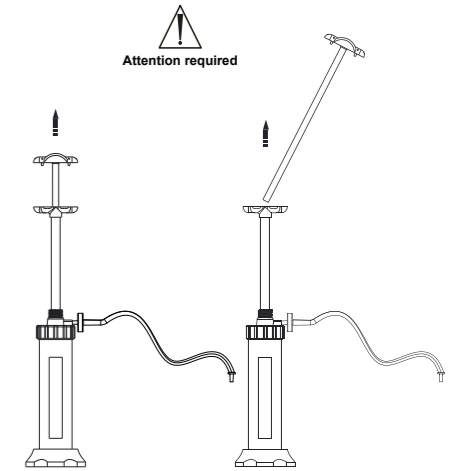⑨


Withdraw the central rod from the mixer plunger.  
The mixer plunger is in two parts: hold the cannula to remove the central rod.  
Caution: keep the mixer plunger's central rod for the final stage of emptying the tube.

⑩

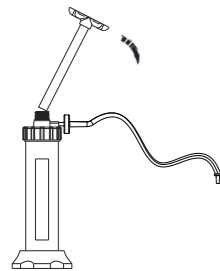
Attention required


Break the mixer plunger tube.  
Caution: the tube must be fully pulled up to be properly broken in two.

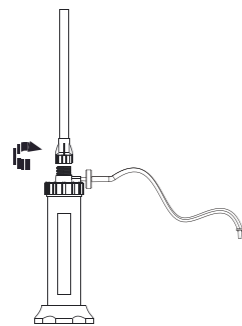⑪


Screw the injection tube onto the mixer.

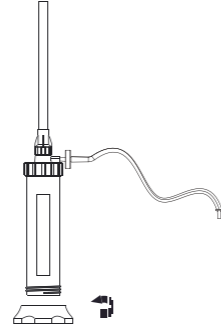⑫


Unscrew the base of the mixer.

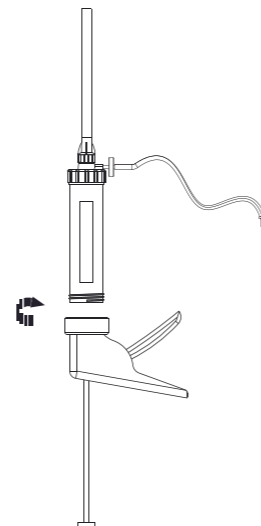⑬


Screw the reservoir onto the injection handset.  
Caution: screw the reservoir on tightly.

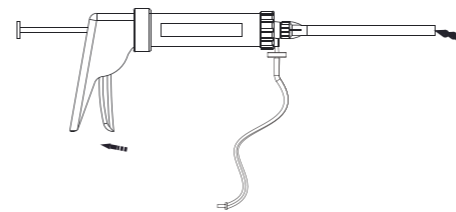⑭


Before injecting, force out any air from the tube by actuating the handle.

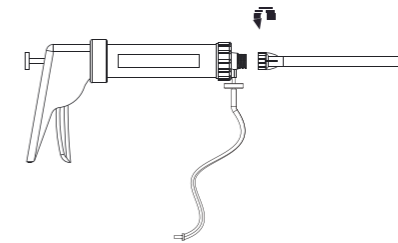⑮

After the cement has been injected, unscrew the injection tube.

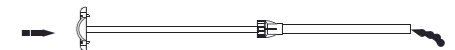⑯

Empty the tube with the mixer plunger's central rod to recover the cement remaining inside the injection tube.

# VACUUKIT®

Vacuum mixing and injection system for bone cement.
Instructions for use

Fabriqu  en France par / Manufactured in France by / In Frankreich hergestellt von / Fabricado en Francia por / Fabricato in Francia da / Fabricado em Frana por / Fransada imal edilmisdir :

**TEKNIMED S.A.S**  
**8, rue du Corps Franc-Pommies**  
**65500 VIC EN BIGORRE - FRANCE**  
T l. (33)5 62 96 88 38  
Fax (33)5 62 96 28 72  
www.teknimed.com

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